

# GO-PAP™

## Disposable CPAP Device


The GO-PAP is a disposable emergency CPAP device that runs off of the barbed valve outlet on an O<sub>2</sub> source. It operates at 10LPM and delivers approximately 30% FiO<sub>2</sub>. There is an adjustable PEEP (5, 7.5, and 10cm H<sub>2</sub>O) valve integrated into the generator. The PEEP is independent of the flow, so you never have to worry about adjusting your flow to maintain your PEEP levels. Additionally, this completely disposable system offers integrated nebulization while maintaining a consistent flow to the patient.



40 Minutes of Run Time \*\*

Delivers ~90LPM of flow

Nebulize with one tank using the Neb-Connect®

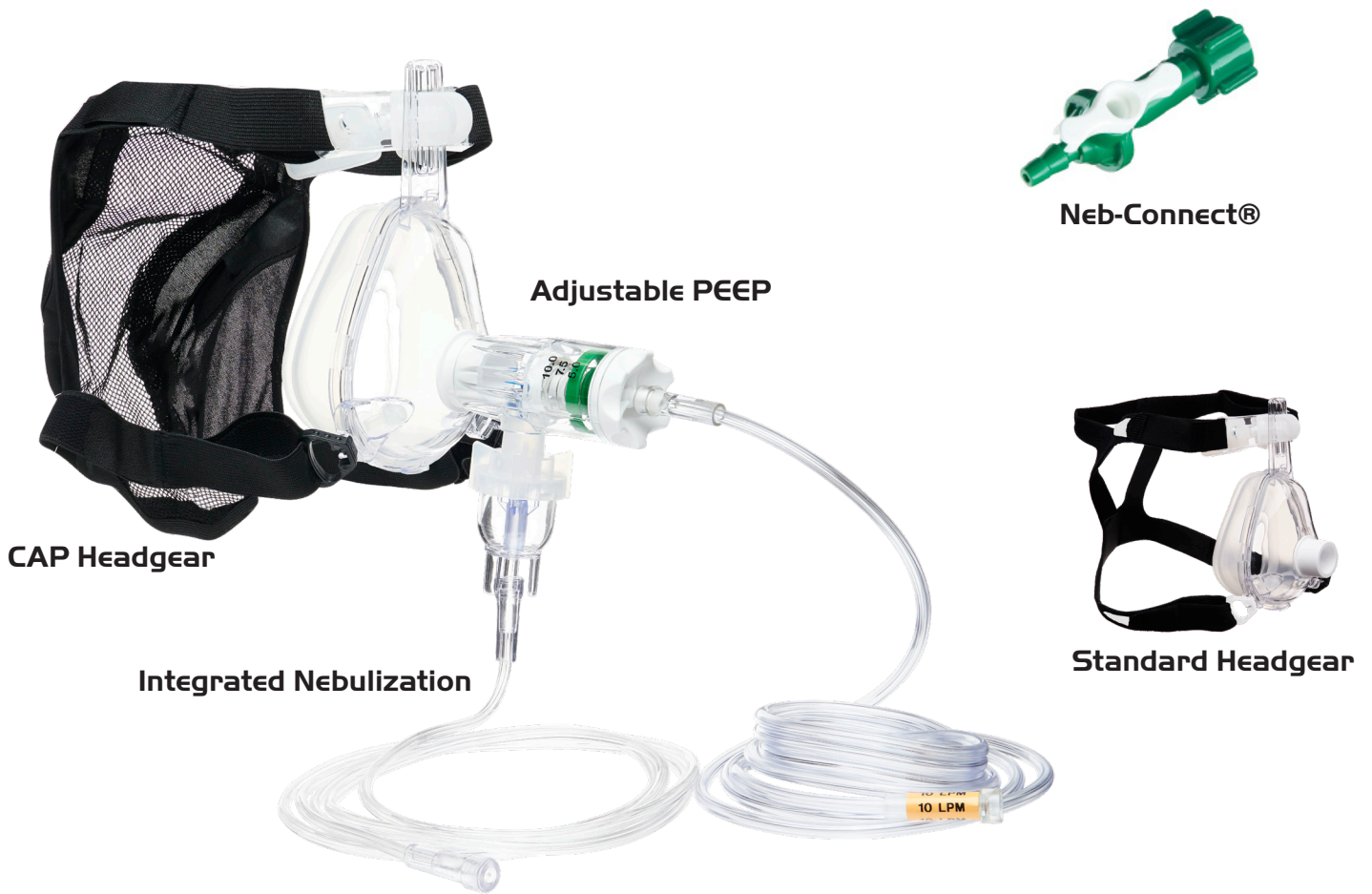

CAP Headgear

Adjustable PEEP



Neb-Connect®



Standard Headgear

Integrated Nebulization

\*\*D-Cylinder Tank

2018F-03-10-2022

## GO-PAP™ System

Part No.	PEEP Setting	Size	FiO <sub>2</sub>	Headgear	Nebulizer	Neb-Connect	Case/Qty.
313-4602NC	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	Standard Headgear	Yes	Yes	10
313-4603NC	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	Standard Headgear	Yes	Yes	10
313-4604NC	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	CAP Headgear	Yes	Yes	10
313-4605NC	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	CAP Headgear	Yes	Yes	10
313-4601N	5/7.5/10cm H <sub>2</sub> O	Adult Small	30%	Standard Headgear	Yes	-	10
313-4602N	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	Standard Headgear	Yes	-	10
313-4603N	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	Standard Headgear	Yes	-	10
313-4604N	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	CAP Headgear	Yes	-	10
313-4605N	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	CAP Headgear	Yes	-	10
313-4601	5/7.5/10cm H <sub>2</sub> O	Adult Small	30%	Standard Headgear	-	-	10
313-4602	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	Standard Headgear	-	-	10
313-4603	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	Standard Headgear	-	-	10
313-4604	5/7.5/10cm H <sub>2</sub> O	Adult Medium	30%	CAP Headgear	-	-	10
313-4605	5/7.5/10cm H <sub>2</sub> O	Adult Large	30%	CAP Headgear	-	-	10

## GO-PAP™ Accessories

Part No.	Description	Size	FiO <sub>2</sub>	Case/Qty.
313-7028	BiTrac ED Mask with Standard Headgear	Adult Small	-	10
313-7029	BiTrac ED Mask with Standard Headgear	Adult Medium	-	10
313-7030	BiTrac ED Mask with Standard Headgear	Adult Large	-	10
313-7031	BiTrac ED Mask with Standard Headgear	Adult Extra Large	-	10
313-7028C	BiTrac ED Mask with CAP Headgear	Adult Small	-	10
313-7029C	BiTrac ED Mask with CAP Headgear	Adult Medium	-	10
313-7030C	BiTrac ED Mask with CAP Headgear	Adult Large	-	10
313-7031C	BiTrac ED Mask with CAP Headgear	Adult Extra Large	-	10
313-011111	Neb-Connect® Standalone	50 PSi DISS port	-	30
313-4600	GO-PAP generator with Oxygen Tubing	N/A	30%	10